# amoena

## **NIPPLE SET**

Gebrauchsanweisung. Instructions for use. Mode d'emploi.  
Instrucciones de uso. Instrução de uso. Istruzioni per l'uso.  
Gebruiksaanwijzing. Brugsanvisning. Instruksjoner for bruk.  
Användningsinstruktioner. Instrukcja użycia. Instrução de uso.  
Käyttöohjeet. Návod k použití. Инструкции по применению.  
Οδηγίες χρήσης. 使用説明. 使用説明書. مادختسالاتاميلعت

Deutsch	DE.....	3
Englisch	ENG .....	5
Français	FR.....	7
Español	ES.....	9
Italiano	IT .....	11
Nederlands	NL .....	13
Dansk	DK .....	15
Norsk	NO .....	17
Svenska	SE.....	19
Polski	PL.....	21
Portugal	PT.....	23
Suomalainen	FI .....	25
Český	CZ.....	27
Русский	RU .....	29
Ελληνικά	GR.....	31
中國	CN .....	33
日本語	JP.....	35
37 .....	ARAB العربية	

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**Thank you for choosing a quality product from Amoena Medizin-Orthopädie-Technik GmbH. Since 1975, Amoena has stood for innovative and high-quality products for the special needs of women who have undergone breast surgery.**

**Indication / Designated Purpose**

Target group: Women after various breast surgeries.

Amoena nipples create a visual balance after breast surgery. The self-adhesive Amoena nipples complete the natural silhouette and provide maximum security combined with exceptional comfort.

**Contraindications/Risks/Side effects**

No contraindications or risks are known or expected when using the above-mentioned products.

A contraindication exists if there is a known allergy to one of the components that may come into contact with the skin. In rare cases, residual risks such as allergic reactions may occur. Please talk to your GP if you experience any significant side effects.

**NIPPLES**

Art. No.	Article
135	Nipple Set Ebony
136	Nipple Set Ivory
137	Nipple Set Almond
138	Nipple Set Bronze
139	Nipple Set Tawny

**Material Composition**

Polydimethylsiloxane

**General Instructions for Use**

- Determine the position for the nipple and place it on the breast form, breast shaper or directly onto your skin. The adhesive material used is long-lasting and skin-friendly.
- It can be removed and reapplied without any problems.
- Thanks to the soft, comfortable material, Amoena nipples adapt perfectly to the natural rounding of the breast or breast form. Any unevenness from putting on the Amoena nipples may be corrected by gentle stroking.

### General Care Instructions

- Careful handling and regular care are crucial for the long-term preservation of the adhesive area of the nipples. Daily cleaning reactivates the adhesive strength and is therefore the most important prerequisite for the secure fit of the nipples.
- The Amoena Soft Cleanser contained in the set is designed for cleaning and enables optimum adhesion. Other cleaning products can reduce the adhesive strength.
- To clean, add a drop of Amoena Soft Cleanser to the previously moistened adhesive surface of the nipple. Then foam up by hand and rinse with plenty of lukewarm water. Now leave the nipple to air dry or use a hairdryer on a low temperature setting.
- Please store the nipple on the carrier film provided.

### Storage instructions and shelf life

Amoena nipples have a shelf life of 6 years (expiry date on label), but they can continue to be used for up to 2 years.

### Special Instructions

Page 40

### Disposal Instructions

Dispose of the products in household waste.

### Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

### Warranty

The statutory warranty obligation applies.

### Service Information

Do you have any questions?

If so, we would like to help – please call us or write to us.

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Amoena is a registered trademark of Amoena Medizin-Orthopädie-Technik GmbH.
© 2024 All rights reserved Amoena Medizin-Orthopädie-Technik GmbH,
Amoena Sweden AB
Amoena UK Ltd